

SPEED ADJUSTMENT FOR VENDLET V5S

With the new speed adjustment for VENDLET V5S it is possible to lower the speed of the bars. In certain situations it may be desirable to slow the bars so that the sheet tightens more slowly onto the bar.

With the new hand control speed can be set at:

- 100 % (normal speed)
- 75 %
- 50 %.

It may be advantageous to slow the bars in situations where the patient:


- Has severe perceptual disruption
- Severe spatial orientation disorders
- Acquired brain injury and neurological disorders
- Severe spasticity
- Several intravenous sites or intubated (most relevant for hospitals).

The hand control with speed adjustment is an optional extra that can be purchased additionally. For all VENDLET V5S systems purchased after June 1, 2016, the new hand control can simply be connected after which speed can be regulated.

For systems purchased before this date it is necessary to purchase a new control unit also, as a new operating system has been developed to control speed regulation.

The new hand control has an inbuilt light. In this way it can also serve as a small torch, allowing night shift personnel to discreetly check that all is well with sleeping patients.

Item no.: 6000042

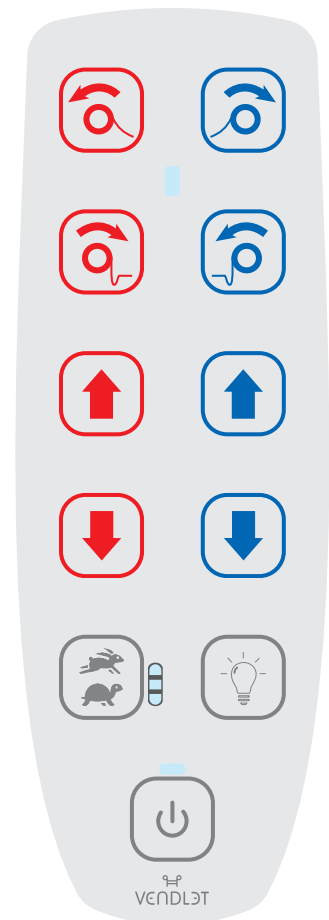

The new hand control for VENDLET V5S allows you to regulate the speed of the motorised bars.